DK .....	Positionerings sejl .....	2
GB/US .....	Positioning Sling.....	5
DE.....	Positionierungstuch.....	9
SE.....	Positionerings Sele.....	12
FR.....	Harnais de positionnement .....	15
IT .....	Imbragatura di posizionamento.....	18
ES.....	Eslinga de posicionamiento .....	21
BR .....	Bolsa de suspensão.....	24

Manual – Version 1.00

Vers. 1.00

**Side positioning**

1 x Positioning sling Item nos: 284226

**Prone positioning**Prone positioning package: Item nos: 284225  
2 x Positioning slings  
2 x Multi Support slings  
1 x Security strap

<b>1.00 Purpose and use .....</b>	<b>5</b>
1.01 Manufacturer .....	5
1.02 Purpose .....	5
1.03 Important/Precautions.....	5
1.04 Use .....	5
<b>2.00 Maintenance .....</b>	<b>6</b>
2.01 Cleaning .....	6
2.02 The owner's daily maintenance duty .....	6
2.03 Disposal of slings .....	6
<b>3.00 Service and lifetime .....</b>	<b>6</b>
3.01 Safety/service inspections .....	6
3.02 Lifetime .....	7
<b>4.00 Technical specifications .....</b>	<b>7</b>
<b>5.00 EC-Declaration of conformity .....</b>	<b>7</b>
<b>6.00 Environmental policy statement – V. Guldmann A/S .8</b>	
<b>WARRANTY – U.S.A and Countries outside the EU .....</b>	<b>8</b>
A. Users guide .....	8
B. WARRANTY .....	8
<b>For instructions on how to use the positioning sling to move a patient sideways .....</b>	<b>27</b>
<b>For instructions on how to the use the positioning sling to move a patient to a face-down position....</b>	<b>37</b>

**1.00 Purpose and use****1.01 Manufacturer**V. Guldmann A/S  
Graham Bells Vej 21-23A  
DK-8200 Aarhus N  
Tel. + 45 8741 3100  
Fax + 45 8741 3131  
www.guldmann.com**1.02 Purpose**The positioning sling is used to turn and position patients at the OR table before and after surgical procedures.  
The sling is designed for use in a ceiling hoist system.**Side positioning**

During side positioning procedures, patients are turned using a positioning sling and moved from lying on their back to lying on their side.

**Prone positioning**

During prone positioning procedures use the package that contains two positioning slings, two Multi support slings and one security strap. The patient is turned from a supine to a prone position by using one positioning sling and then back to a supine position using the other positioning sling following the operation.

Once the patient has been turned over and is in the prone position, it is possible to lift parts of the body using the accompanying Multi support sling, so that positioning pillows can be placed underneath the patient prior to the operation.

**Conditions for use**

The use of the sling is subject to the following:

- The sling is used by trained staff or persons who have been instructed in the use of the sling in question.
- The safe working load of 255 kg must never be exceeded.
- The sling is used to turn a person in a lying position.
- The helper pays attention to the well-being of the user when using the sling.
- The sling is used with the Guldmann lifting hanger.

**1.03 Important/Precautions**

- Read the instructions carefully before using the sling.
- The sling's maximum load must never be exceeded.
- The sling may only be used to roll or support a person.
- Before a sling is used, it must be examined according to point 2.02.
- Possible repairs must only be made by the manufacturer.

**1.04 Use***If there is any doubt about the selection or use of a lifting sling, please contact your supplier.***Important!**

Plan the move. Never leaving the user in the lifting sling unattended. Do not start to lift until it has been checked that the user cannot get trapped and that the sling does not catch on the bed, wheelchair or other obstacles. The user's head, arms, hands and feet must not be in danger of becoming trapped. Be careful with any tubes and wires that are attached to the user and/or equipment. Check that the hand control and hand control cable is free of hanger, patient and other object before the hoist is activated up or down moved.

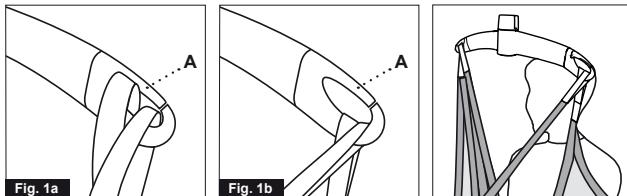
Guldmann shall not be liable for faults or accidents due to incorrect use of the lifting sling, or for reasons of inadequate attention

on the part of the carer or user. If the sling is used in combination with products that are not manufactured by Guldmann, a risk assessment must be made by qualified staff.

### GH Lifting hanger

#### Caution!


**Be careful when attaching the lifting sling on the hooks.**  
**Check that the straps have been pulled completely through the rubber safety catch (A) and into place in the lifting hanger's hooks. When pressing the up button to lift the user, check again that all the straps remain correctly placed in the lifting hanger's hooks (fig. 1a and fig. 1b).**

### GH lifting hanger, 4 attachment points

#### Caution!

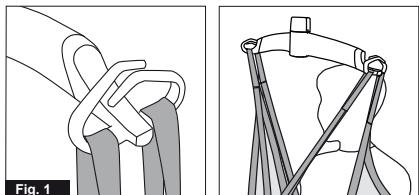**Be careful when attaching the lifting sling's straps on the hooks. Check that the straps have been correctly placed in the lifting hanger's hooks. When pressing the up button on the hand control to lift the user, check again that all straps remain correctly placed in the lifting hanger's hooks (Fig. 1).**

For instructions on how to use the positioning sling for side positioning procedures, see page 27.

For instructions on how to use the positioning sling for prone positioning procedures, see page 37.

---

## 2.00 Maintenance

### 2.01 Cleaning

Normal washing at the indicated temperature  
 The sling can be washed a maximum of 5 times

Do not use bleaching agent

Tumble-drying at low temperature

Do not iron

---

### 2.02 The owner's daily maintenance duty

Check the lifting sling for wear and damage before use according to the following checklist which is not intended to represent all potential inspection steps. Potential damage may vary. Judgment of inspector/site prevails.

### Sling inspection checklist

Before using a Guldmann sling / accessory check the following:

#### Is the sling clean?

Follow facility specific infection control procedure.

#### Is the sling's label present, legible and complete?

Missing, illegible or incomplete sling label(s) could make identification of appropriate size of the sling, function of sling, and or weight limit capacity of the sling impossible.

#### Are the lifting straps and stitches intact?

- Look for broken or worn stitches
- Look for knots in straps
- Look for tears or fraying of straps
- Look for snags or punctures or holes
- Look for any particles in fabric or straps

#### Is the fabric intact?

- Look for abnormal wear patterns, excessive wear, abrasive evidence
- Look for cuts or frayed fabric
- Look for unusual or significant discoloration
- Look for snags, punctures, tears, holes
- Look for frayed or insecure seams
- Look for any acid / caustic / thermal burns
- Look for changes in material consistency, e.g. increased stiffness
- Look for any imbedded particles

Are slings the original size and length without the use of knots, pins, tape or other methods to change the shape, shorten or lengthen them?

### Conclusion

If the sling suffers from one or more of the above mentioned conditions then it must be taken out of service regardless of the weight of the person to be lifted.

---

## 2.03 Disposal of slings

Slings are disposed of by incineration. By proper incineration polyester will be degraded to carbon dioxide and water.

---

## 3.00 Service and lifetime

---

### 3.01 Safety/service inspections

In accordance with international standard EN/ISO 10535 "Hoist for the transfer of disabled persons – Requirements and test methods" an inspection must be performed every 6-month according to the following instructions, which is not intended to represent all potential inspection steps. Potential damage may vary. Judgment of inspector/site prevails.

#### Safe Operating Practices with Slings

Considerations for damaged or defective slings and taking them out of service:

Withdraw the sling from service if one or more of the following conditions are present:

- chemical or caustic burns
- melting or charring of any part of the sling
- snags, punctures, tears or cuts
- broken or worn stitches
- missing, illegible or incomplete sling tag
- knots in any part of the sling
- abrasion
- other visible damage that causes doubt as to the strength of the sling

Sling inspection is done for the protection of the user, the caregiver, and the overall hospital site safety. A sling inspection system has additional benefit. Systematic sling inspection will assist in the identification of damage trends, potentially leading to cost effective suggestions and results. The inspection process can also help to identify inventory duplicity in certain sling types and sizes.

### **Sling inspection system**

Development of a specific procedure and program for the inspection of slings at your facility is your best safeguard. Consider employing a three part system of inspection. Slings that are removed from service and are not capable of repair should be disposed of so they are unfit for any future use and can not find a way back into active inventory.

#### **1) Initial**

This level of inspection is done at the time that the sling is received into your facility. The inspector should ensure that no damage has occurred during transit, and also verify that the sling work load limits match those contained in the manufacturer's catalogue. If your facility documents the sling inspection process through written inspection records, the paper trail should begin at this stage.

#### **2) Frequent**

The frequent level of inspection should be done by the sling user before each use. The sling should be examined and removed from service if damage is detected. The sling user should also determine that the sling is proper for the user conditions, care task required and the required weight capacity.

#### **3) Periodic**

Your facility might want to consider implementing a program for a periodic level of inspection at regular intervals. The interval should be based upon the frequency of use, severity of the service cycle and information derived through the inspection process. Recommendations to prevent damage and enhance service life could be made by staff that perform the periodic inspections. If written inspection records are maintained, they should always reference the unique sling identification number, and be updated to record the condition of the sling. Not intended to represent all potential inspection steps or all potential aspects of product management program. Judgment of inspector/site prevails.

### **Sling inspection technique**

The sling inspection procedure should be thorough, systematic and consistent; both visual and "hands on" inspection techniques are recommended. Certain forms of damage are far more discernable through hands-on inspection, than by visual inspection. For example, fabric stiffness, crushed webbing, as well as, thinning fabric can be identified through tactile inspection. Visual inspection alone may not reveal all forms of sling damage. Once signs of damage have been identified, do not downgrade the work load limit of the sling, with the intent of continuing to use it, but at limited capacity or frequency. This is sometimes done to get more service life out of a damaged sling. The operating rule and standard should be: intact = use; damage = do not use.

Consider the practice of documenting sling inspections through written inspection records. The documentation should include information such as: the name of manufacturer, the sling stock number, width and length, the unique sling identification number (important in differentiating similar slings), as well as the condition of the sling. Other important information might also include the date it was received or put into use at your facility and any special features (if applicable). A beneficial outcome of an inspection program would be the realization of repetitive forms of damage and the analysis that would lead to specific recommendations.

### **Sample visual examples of synthetic sling damage<sup>x)</sup>**

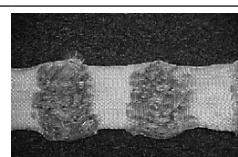
#### *Chemical/caustic burns*

#### *Broken stitching*


#### *Crushed / Frayed webbing*

#### *Knots*
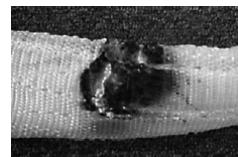
#### *Melting / Charring*

<sup>x)</sup> sample visual images not intended to represent all types of potential damage

### **3.02 Lifetime**

The life of the sling is individual and depends on how it is used, washed etc. Before use the sling must be examined according to description in section 2.02 and if it does not meet the inspection requirements, it must be discarded.

The sling can be washed a maximum of 5 times. Remember to mark off the label following each wash. The sling must be disposed of after 5 washes.

### **4.00 Technical specifications**

Lifting capacity, SWL .....	..... 255 kg
Material, white part: .....	Polyester
Sliding material .....	Nylon

### **5.00 EC-Declaration of conformity**

The product is manufactured in compliance with the Council Directive 93/42/EEC of June 14th 1993, including amendments, as medical device class 1.

## 6.00 Environmental policy statement – V. Guldmann A/S

Guldmann is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

It is Guldmann's goal to:

- Comply with the current environmental legislation (e.g. WEEE and REACH directives)
- Ensure that we, at the widest possible range, use RoHS compliant materials and components
- Ensure that our products do not have an unnecessary negative impact on the environment regarding use, recirculation or disposal
- Ensure that our products contribute to a positive working environment in the places they are utilised

Inspections are made annually by the Department for Nature and Environment from the Municipality of Aarhus using the Danish Environmental Protection Act, section 42 as a reference.

## WARRANTY – U.S.A and Countries outside the EU

### A. Users guide

Before using the product, read the entire operation manual including warranty.

### B. WARRANTY

Guldmann warrants its equipment is free from material defects under normal use, and will perform substantially in accordance with the specifications set forth in documentation provided with the equipment.

This express warranty shall be in effect for one year from the date of original purchase and installation (the "Warranty Period"). If a valid claim is made during the Warranty Period for malfunction or equipment defect, Guldmann will repair or replace the equipment at no additional cost to you. Guldmann retains sole discretion as to whether the equipment will be repaired or replaced.

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann designated technician. Any parts or components repaired or replaced by a Guldmann designated technician will be guaranteed for the remainder of the Warranty Period.

The warranty does not cover any part of the equipment which has been subject to damage or abuse by the user or others. The warranty does not cover any part of the equipment which has been altered or changed in any way by the user or others. Guldmann does not warrant that the lifting device functions will meet your requirements, be uninterrupted or error free.

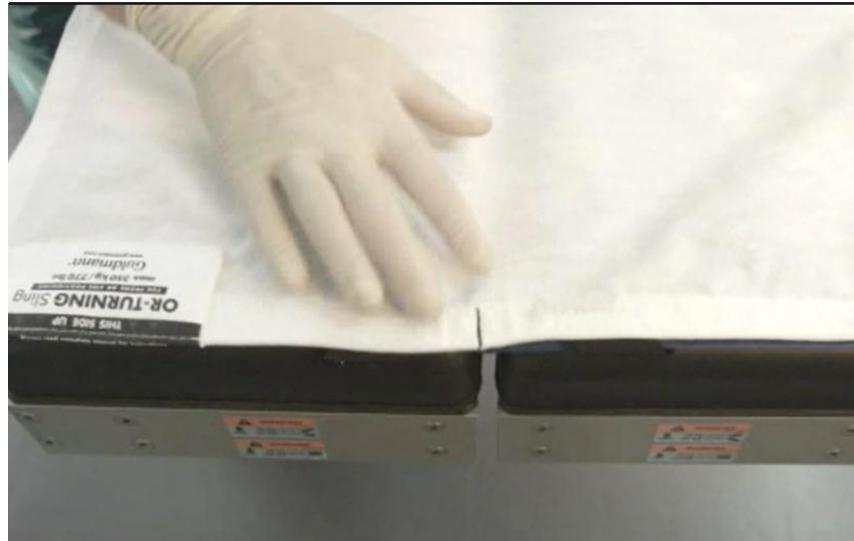
The warranty set forth is in lieu of all other express and implied warranties, whether oral, written or implied, and the remedies set forth above are your sole and exclusive remedies. Only an authorized officer of Guldmann may make modifications to this warranty, or additional warranties binding on Guldmann. Accordingly, additional statements such as advertising or presentations, whether oral or written, do not constitute warranties by Guldmann.

#### Service or Repair

Contact Guldmann Repair for an authorization to return any defective item during the Warranty Period. You will be provided with a return authorization number and address for returning the item for warranty service or replacement. Do not return items to Guldmann under warranty without receiving a Return Authorization Number.

If mailing the item, pack it carefully in a sturdy carton to prevent damage. Include your Return Authorization Number, a brief description of the problem and your return address and phone number. Guldmann does not assume the risk of loss or damage while in transit, so it is recommended you insure the package.

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## 1. GB/US

### **Preparation for side positioning**

If a positioning bolster will be used during procedure, place the Multi Support sling **under** the OR Turning sling **before** it is placed on the OR table.

The Multi support sling must be places at the chest and below the axilla. (see photo 9 on page 36 for proper positioning).

#### **Important!**

The center mark must always be face-up and positioned in the middle. White and soft side face upwards. The positioning sling's side border with the long folded white and blue edge should be on the side the patient is turning **toward** and will lay on.

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## 5. GB/US

The positioning sling's lifting straps are now placed onto the lifting hanger. Ensure that the patient's "anaesthesia arm" is on the same side as the lifting straps. By doing so, you can ensure that this arm is always on top during the turning procedure, so that tubes and cables do not become tangled or caught. Loosen the 4 elastic bands from the corners.

### Important!

Remember to check the elastic bands to ensure they have been loosened from all 4 corners.

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## 2. GB/US

Position and fasten the bottom part of the two-piece positioning sling to the OR table by fixing the positioning sling's 4 elastic bands around the 4 corners of the OR table.

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### 3. GB/US

Should it become necessary to lift the patient's upper body when placing bolsters, place the two Multi support slings over the bottom positioning sling at the patient's upper body so that they support the pelvis and abdomen, as well as at the patient's chest below the axilla (see photo 9 on page 47 for proper positioning).

The patient's arms must be outside of the sling.

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## 4. GB/US

Attach the top half of the two-piece positioning sling in the same way as the bottom half by fixing the 4 elastic bands around the OR tables 4 corners. Note however that the two lifting straps must now be placed on the opposite side.

### Important!

Always utilize the necessary safety steps to accompany use of the sling, such as correct number of caregivers and use of OR table side stops. There must be mounted sidestops on each side of the OR table.

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## 6. GB/US

When the patient is ready for turning, the upper positioning sling's lifting straps are now placed onto the ceiling hoist's hanger. Ensure that the patient's "anaesthesia arm" is on the same side as the lifting straps. By doing so, you can ensure that this arm is always on top during the turning procedure, so that tubes and cables do not become tangled or caught. Loosen the 4 elastic bands from the corners.

### Important!

Remember to ensure that only the elastic bands belonging to the top sling are loosened. These 4 bands are all of the same colour, though their colour is different from the bands on the bottom sling (the top sling's elastic bands are either white or black, depending on which side the turning procedure is being done on).

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## 7. GB/US

### **Important!**

Do not push the patient before the "side stoppers" have been fitted onto the OR table. Push the patient's hip and shoulders while simultaneously activating the lift, so that the hanger and the mounted positioning sling are moved upwards. The turning procedure has now begun.

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## 8. GB/US

When the patient has been turned over and is lying prone position, manually remove the sling by pulling it out piece by piece, material towards material. Avoid creating friction against the patient's skin.

## 8. DE

When the patient has been turned over and is lying prone position, manually remove the sling by pulling it out piece by piece, material towards material. Avoid creating friction against the patient's skin.

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## 9. GB/US

If necessary, the patient's arms can be placed over his/her head. Adjust the two Multi support slings so that they are optimally positioned. Place the straps onto the lifting hanger. Be aware that the hanger is placed parallel with the patient's body.

Lift the patient's upper body and place the bolster / bolsters or other device. Lower and place the patient as required. The Multi support sling stays under the patient during the procedure. Remove the straps from the lifting hanger.

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## 11. GB/US

### Prone positioning – turning the patient from a prone to a supine position

Remove the security strap before moving the patient back to the supine position. Loosen the 4 elastic bands from the corners. Reattach the Multi support slings to the lifting hanger, lift the patient, and remove the bolsters. Lower the patient onto the bottom half of the two positioning slings.

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## 13. GB/US

Prepare the patient for turning supine. The patient can be turned by pushing again on his/her hip and shoulders while simultaneously activating the lift.

**Important! ”Side stoppers” must be fitted onto the OR table.**

Manually pull out the positioning slings and the two Multi support slings. Take care to avoid causing friction against the patient's tissue. Ensure that the patient is lying correctly and comfortably, and without any entanglements.

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**Guldmann Srl**
**Guldmann Inc.**